

ECG DIAGNOSTIC ELECTRODE

MR CONDITIONAL




FOR USE WITH:

Product No	Reference No	Product No	Reference No
6500009V	A10004-5	32029078	OTL-5235-RT-5
6500011V	A10004-60	32029076	TSL-4532-5
6500026V	A10006-5RT		
6500002V	A10006-1-60RT		
6500012V	A10004-60T		
6500010V	A10004-5T		


DURATION OF USE

● < 24 hours


INSTRUCTIONS FOR USE

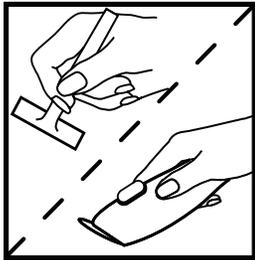


● Shave/Clip

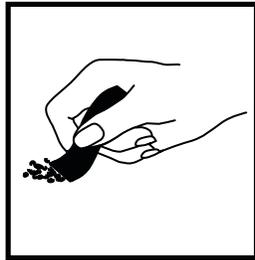


● Prep

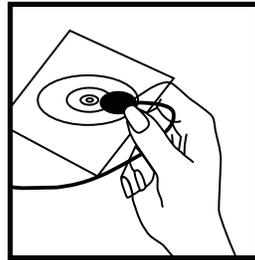

● Connect

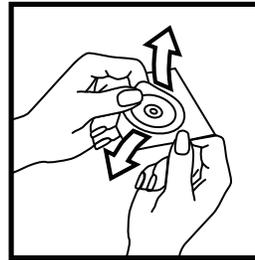

● Peel

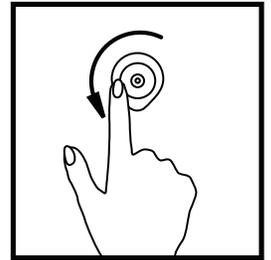

● Apply

CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML171 (04/2021)