


## ECG DIAGNOSTIC ELECTRODE


### FOR USE WITH:

Product No	Reference No	Product No	Reference No	Product No	Reference No
6500472V	A10005-10P	6500040V	A10012-60S	6500038V	A10012-6S
6502923V	A10015-10S	6500001V	A10006-1-60TS	6500076V	A10084-4
6502764V	A10101-10	6500027V	A10006-5T	6500065V	A10043
6500013V	A10005	6500073V	A10072-1-60	32029080	OTL-4227-S-5
6500014V	A10005-1-60	6500590V	A10072-3	32029079	OTL-5235-5
6500016V	A10005-4NB	6502800V	A10072-5	32029077	OTL-5235-S-5
6500018V	A10005-60	6500509V	A10009-100F	6502681V	Z05-5
6500020V	A10005NB	6500041V	A10013-100S	6502682V	Z05-50
6500022V	A10006-1-60S	6500066V	A10049		
6500024V	A10006-5	6500033V	A10009-100		
6500481V	A10005-7NB	6500032V	A10009		
6500030V	A10007-5	6500006V	A10003-4		
6500036V	A10012-4-60S	6500008V	A10003-4-60		
6500037V	A10012-5S				


### INSTRUCTIONS FOR USE

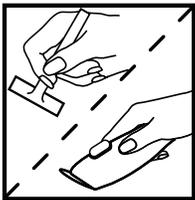


• Shave/Clip

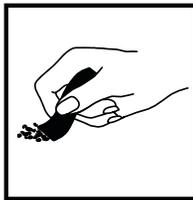


• Prep

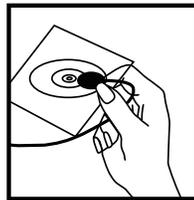

• Connect

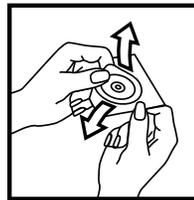

• Peel

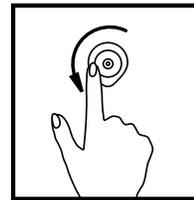

• Apply

### DURATION OF USE



• < 24 hours

### CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

### PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML168 (04/2021)



HS.NisshaMedical.com/IFU



Not made with natural rubber latex

