


RESTING ECG TAB ELECTRODE

MR CONDITIONAL




FOR USE WITH:

Product No	Reference No
6500082V	A10201
6500083V	A10201-500
6503180V	A10202
6503179V	A10202-10
6500045V	A10024

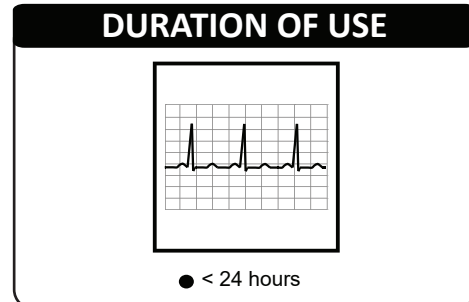

INSTRUCTIONS FOR USE

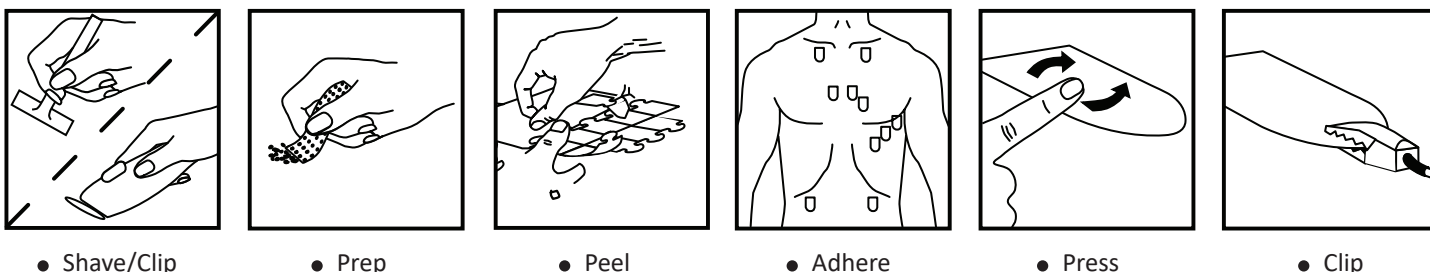

CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML169 (04/2024)



HS.NisshaMedical.com/IFU



Not made with natural rubber latex

