

Multi-function Defibrillation Pads

Monophasic or Biphasic Compatible


Manual Defibrillation • Cardioversion • Pacing • ECG Monitoring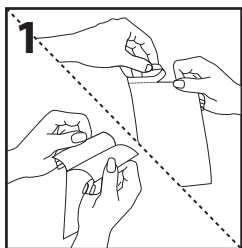


1. Remove Electrodes from pouch.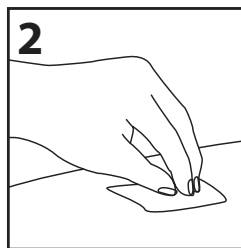


2. Ensure skin site is clean and dry.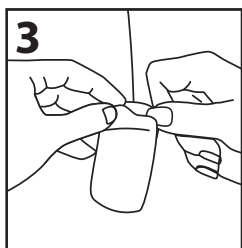


3. Remove Electrodes from release liner.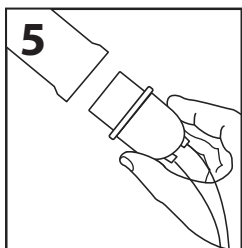


5. Connect electrodes to manual defibrillator.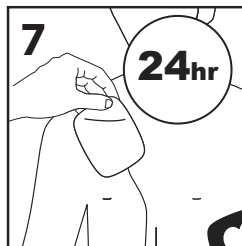

7. Apply a new set of electrodes after: 24 hours on skin or 50 defibrillation shocks.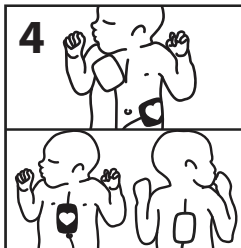

4. Apply electrodes firmly to patient's skin.

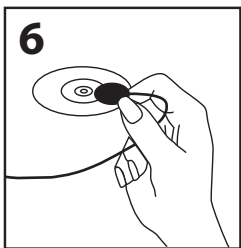6. Connect ECG leads for demand pacing only.

NISSHA
MEDICAL TECHNOLOGIES

HS.NisshaMedical.com/Defib

MANUAL DEFIBRILLATOR

≤ 10Kg / 22lbs
≤ 100 J

Do not delay therapy if unsure of age or weight. Placement of pads varies due to patient size. Refer to placement diagram.

Graphic Controls Acquisition Corp.
d.b.a. Nissha Medical Technologies
400 Exchange Street, Buffalo, NY 14204
HS.NisshaMedical.com

VML195 Rev A
Rev Date: 08/06/25

Not made with
natural rubber latex

10°C
50°F

Rx ONLY

Pediatric Defibrillator Electrodes

Indications For Use (Prescription Use Only)

The HeartSync Infant/Pediatric Radiotranslucent Multifunction Electrodes are indicated for use in external pacing, defibrillation, and monitoring applications as a non-sterile, disposable device for single patient use only. The electrodes provide the conductive interface between the defibrillator and/or the external transcutaneous (noninvasive) cardiac pacemaker and the patient's skin. The electrode is intended for use on pediatric patients whose weight is less than 10 kg. When a patient requires defibrillation, cardioversion or external pacing, these electrodes will be applied to the patient and connected to the instrument. This device is intended for use with defibrillators in manual mode, that are capable of reducing energy below 100 joules. Follow site specific protocols regarding patient weight and energy delivery. In accordance with American Heart Association (AHA) guidelines, "It is reasonable to use an initial dose of 2–4 J/kg of monophasic or biphasic energy for defibrillation. For refractory VF, it may be reasonable to increase the defibrillation dose to 4 J/kg. For subsequent energy levels, a dose of 4 J/kg may be reasonable, and higher energy levels may be considered, though not to exceed 10 J/kg." **Not intended for AED use.**

Compatible Manual Defibrillation Models:

Physio Control: LifePak-15
LifePak-20e

Zoll Medical: R-Series X-Series
Propaq MD

For Manual Defibrillators:

Manual Defibrillators can be used for monitoring, pacing, cardioversion, as well as defibrillation. When used for defibrillation, these electrode pads are for treating patients in cardiopulmonary arrest who are:

- Unconscious,
- Not breathing spontaneously
- Without circulation (without a pulse).

The pads are single use and intended to be used in conjunction with a manual external defibrillator to monitor and deliver defibrillation energy to the patient. The pads are used on patients less than 10 kg or 22 pounds. DO NOT DELAY THERAPY IF YOU ARE NOT SURE OF EXACT AGE OR WEIGHT.

Instructions For Use

1. Remove Electrodes from pouch.
2. Ensure skin site is clean and dry.
3. Remove Electrodes from release liner.
4. Apply electrodes firmly to patient's skin.
5. Connect electrodes to manual defibrillator.
6. Connect ECG leads for demand pacing only.
7. Apply a new set of electrodes after: 24 hours on skin or 50 defibrillation shocks.

Warnings

- Do not use if conductive polymer gel is dried or damaged.
- Misuse of electrodes may cause patient burns.
- Do not use if electrodes are damaged.
- Not compatible with all models. Check compatibility before use.
- Do not use in the presence of magnetic resonance imaging (MRI) devices.
- Avoid touching the patient or other conductive material before/during defibrillation to avoid accidental shock.
- Keep electrodes clear of other electrodes or metal parts in contact with the patient.
- Electrodes should not be repositioned following application.
- Check connector prior to use, do not use if damaged.
- Do not use if beyond expiration date.

Cautions

- Do not open package until immediately prior to using electrodes.
- Electrodes may dry out when removed from packaging and exposed to air.
- If electrodes do not adhere properly to patient, apply a new pair.
- Do not apply any substance to the skin surface that will leave residue.
- Do not apply electrodes over broken or damaged skin.
- Do not crush, fold or store package under heavy objects.
- Pacing requires separate leads and electrodes for monitoring.
- The adhesive foam may be a mild irritant to the skin.
- Electrode placement diagrams are suggestion only. Follow hospital protocol.
- Pacing, defibrillation or cardioversion may cause reddening of the skin.

Any serious incident that has occurred in relation to the defibrillator electrodes should be reported to the manufacturer and the competent authority of the state in which the user and/or patient is established.

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