



12-Lead ECG Electrode Placement

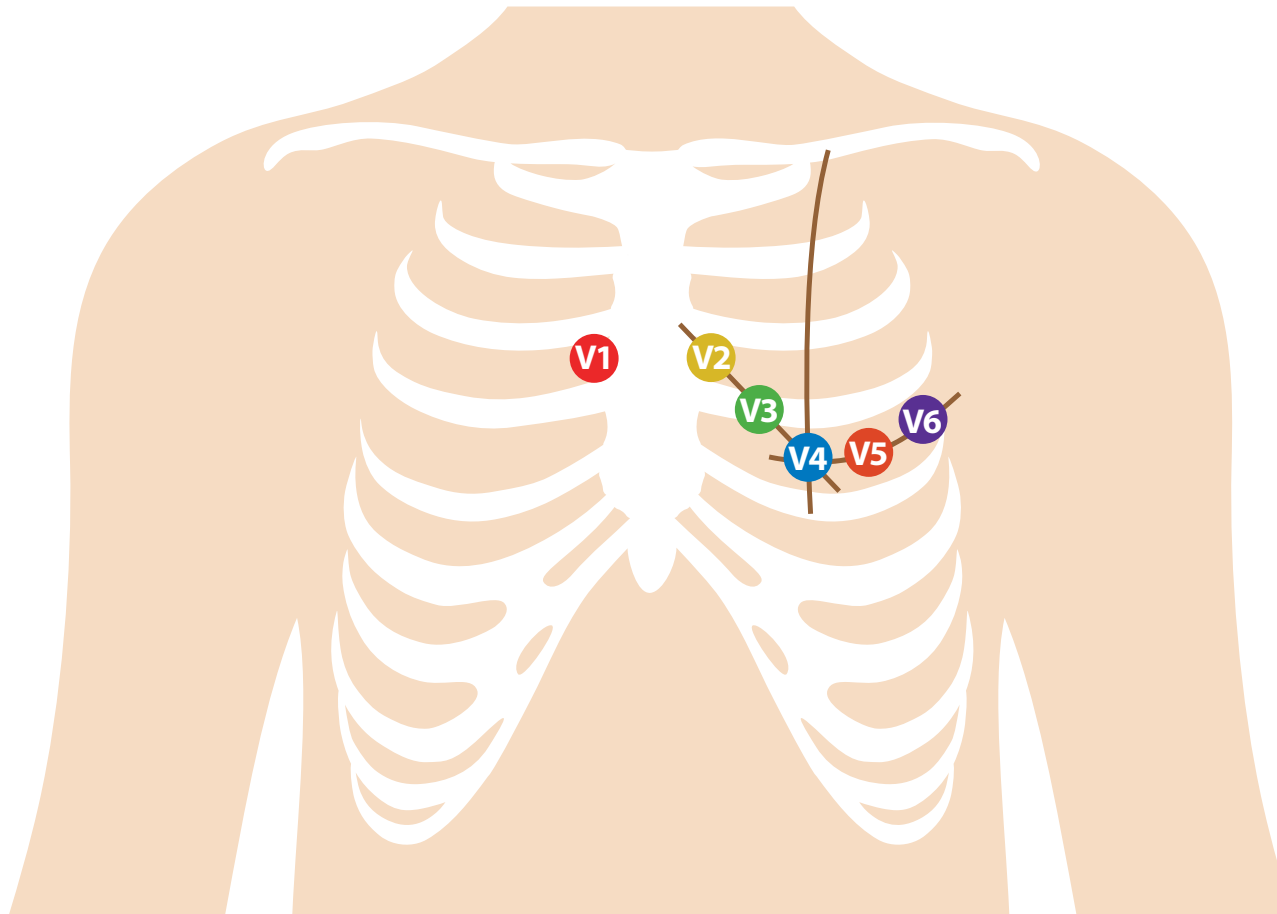

- V1** - Fourth intercostal space on the right sternum
- V2** - Fourth intercostal space at the left sternum
- V3** - Midway between placement of V2 and V4
- V4** - Fifth intercostal space at the midclavicular line
- V5** - Midway between placement of V4 and V6
- V6** - Horizontal to V4, midaxillary line

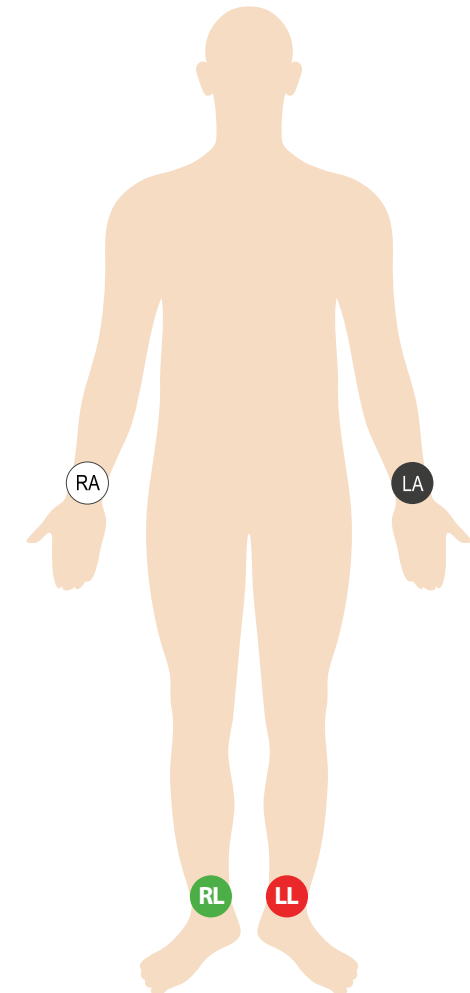

- RA** - Right forearm or wrist
- LA** - Left forearm or wrist
- RL** - Right lower leg to ankle
- LL** - Left lower leg to ankle