

Medtronic Physio-Control

Life-Pack 10P, 12, 15 and 20 - 20ms Pulse Width


Maximum Pacing Duration

Recommended Maximum Pacing Duration for HeartSync Electrodes (in Hours)

	Pacing Rate (ppm)														
	40	50	60	70	80	90	100	110	120	130	140	150	160	170	
Pulse Amplitude (mA)	10	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	20	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	30	24	24	24	24	24	23	20	16	16	14	12	10	8	8
	40	24	24	24	22	22	18	18	16	14	14	10	10	8	8
	50	24	24	22	20	18	16	16	14	14	10	10	8	8	6
	60	24	22	20	18	16	14	14	11	10	9	8	6	5	5
	70	24	23	20	17	15	14	12	11	9	8	6	5	5	4
	80	22	21	20	17	14	13	12	11	8	7	5	4	4	4
	90	20	20	18	16	14	13	11	9	8	6	4	4	3	3
	100	18	18	17	16	13	12	10	9	7	5	4	4	3	3
	110	18	17	16	14	12	10	9	8	6	4	4	3	3	3
	120	16	16	15	13	11	10	8	8	5	4	3	3	2	2
	130	14	14	13	11	10	9	8	7	5	4	3	2	2	2
	140	12	12	11	10	9	8	7	6	4	4	3	2	2	2
	150	11	10	8	6	6	5	5	4	4	3	2	2	2	2
	160	10	8	6	5	4	3	3	3	2	2	2	2	1	1
	170	8	8	4	3	3	3	2	2	2	2	2	1	1	1
	180	8	7	4	3	3	2	2	2	2	1	1	1	1	1
	190	8	6	5	4	3	2	2	2	1	1	1	1	1	1
	200	8	6	4	3	3	2	2	1	1	1	1	1	1	1

Not to be reproduced without permission from Nissha Medical Technologies

Please contact Nissha Medical Technologies at 1.800.669.6905 or HS-Info@NisshaMedical.com for any questions regarding electrode compatibility,

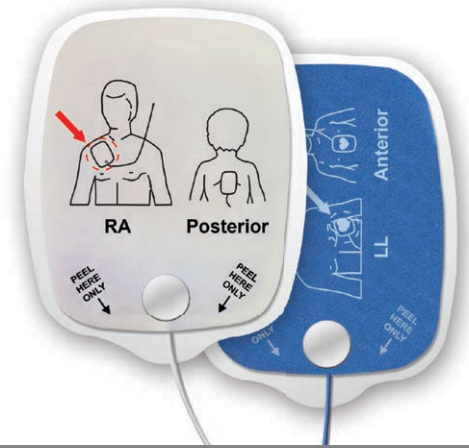

Zoll
 40ms Pulse Width

Maximum Pacing Duration

Recommended Maximum Pacing Duration for HeartSync Electrodes (in Hours)

		Pacing Rate (ppm)															
		30	40	50	60	70	80	90	100	110	120	130	140	150	160	170	180
Pulse Amplitude (mA)	10	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	20	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	30	24	24	22	20	16	14	12	10	8	8	7	6	6	5	5	4
	40	22	16	14	12	10	8	8	6	6	7	6	5	4	4	4	3
	50	20	15	14	12	10	8	7	6	5	4	4	3	3	3	2	2
	60	16	14	12	10	9	7	6	5	4	4	4	3	3	2	2	2
	70	14	12	10	10	8	6	5	5	4	3	3	3	2	2	2	2
	80	12	10	9	9	8	6	5	4	3	3	3	2	2	2	2	2
	90	10	9	8	8	6	4	4	4	3	3	2	2	2	2	1	1
	100	9	8	7	6	5	4	3	3	3	2	2	2	2	2	1	1
	110	8	7	6	6	4	3	3	3	2	2	2	1	1	1	1	1
	120	8	6	6	5	4	3	2	2	2	2	2	1	1	1	1	1
	130	7	5	5	5	3	2	2	2	2	1	1	1	1	1	1	1
	140	6	5	4	4	3	2	2	2	2	1	1	1	1	1	1	1
	150	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
	160	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
	170	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
180	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	
190	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	
200	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	

Not to be reproduced without permission from Nissha Medical Technologies

Please contact Nissha Medical Technologies at 1.800.669.6905 or HS-Info@NisshaMedical.com for any questions regarding electrode compatibility,



Philips Codemaster

20ms Pulse Width


Maximum Pacing Duration

Recommended Maximum Pacing Duration for HeartSync Electrodes (in Hours)

	Pacing Rate (ppm)															
	40	50	60	70	80	90	100	110	120	130	140	150	160	170	180	
Pulse Amplitude (mA)	10	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	20	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	30	24	24	24	24	24	23	20	16	16	14	12	10	8	8	8
	40	24	24	24	22	22	18	18	16	14	14	10	10	8	8	8
	50	24	24	22	20	18	16	16	12	12	10	10	8	8	6	6
	60	24	22	20	18	16	14	14	11	10	9	8	6	5	5	5
	70	24	23	20	17	15	14	12	11	9	8	6	5	5	4	4
	80	22	21	20	17	14	13	12	11	8	7	5	4	4	4	3
	90	20	20	18	16	14	13	11	9	8	6	4	4	3	3	2
	100	18	18	17	16	13	12	10	9	7	5	4	4	3	3	2
	110	18	17	16	15	12	10	9	8	6	4	4	3	3	3	2
	120	16	16	15	13	11	10	8	8	5	4	3	3	2	2	1
	130	14	14	13	11	10	9	8	7	5	4	3	2	2	2	1
	140	12	12	11	10	9	8	7	6	4	4	3	2	2	2	1
	150	11	10	8	6	6	5	5	4	4	3	2	2	2	2	1
	160	10	8	6	5	4	3	3	3	2	2	2	2	1	1	1
	170	8	8	4	3	3	3	2	2	2	2	2	1	1	1	1
	180	8	7	4	3	3	2	2	2	2	1	1	1	1	1	1
	190	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
200	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	

Not to be reproduced without permission from Nissha Medical Technologies

Please contact Nissha Medical Technologies at 1.800.669.6905 or HS-Info@NisshaMedical.com for any questions regarding electrode compatibility,

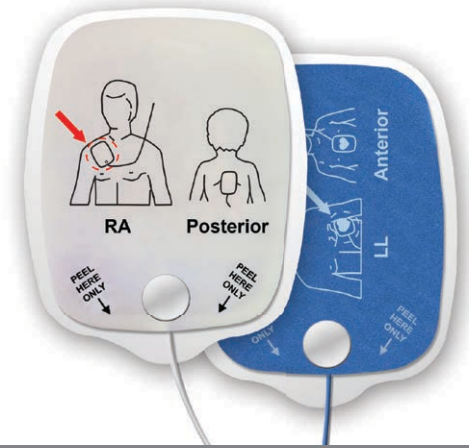

Philips HeartStart MRx
 Monitor/Defibrillator/Pacemaker - 40ms Pulse Width

Maximum Pacing Duration

Recommended Maximum Pacing Duration for HeartSync Electrodes (in Hours)

	Pacing Rate (ppm)																
	30	40	50	60	70	80	90	100	110	120	130	140	150	160	170	180	
Pulse Amplitude (mA)	10	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	20	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24	24
	30	24	24	22	20	16	14	12	10	8	8	7	6	6	5	5	4
	40	22	16	14	12	10	8	8	6	6	7	6	5	4	4	4	3
	50	20	15	14	12	10	8	7	6	5	4	4	3	3	3	2	2
	60	16	14	12	10	9	7	6	5	4	4	4	3	3	2	2	2
	70	14	12	10	10	8	6	5	5	4	3	3	3	2	2	2	2
	80	12	10	9	9	8	6	5	4	3	3	3	2	2	2	2	2
	90	10	9	8	8	6	4	4	4	3	3	2	2	2	2	1	1
	100	9	8	7	6	5	4	3	3	3	2	2	2	2	2	1	1
	110	8	7	6	6	4	3	3	3	2	2	2	1	1	1	1	1
	120	8	6	6	5	4	3	2	2	2	2	2	1	1	1	1	1
	130	7	5	5	5	3	2	2	2	2	1	1	1	1	1	1	1
	140	6	5	4	4	3	2	2	2	2	1	1	1	1	1	1	1
	150	5	4	4	3	3	2	2	2	1	1	1	1	1	1	1	1
	160	4	4	3	3	3	2	1	1	1	1	1	1	1	1	1	1
	170	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
180	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	
190	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	
200	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	

Not to be reproduced without permission from Nissha Medical Technologies

Please contact Nissha Medical Technologies at 1.800.669.6905 or HS-Info@NisshaMedical.com for any questions regarding electrode compatibility,