



tissueLINK™  
Topical Skin Adhesive

**FEATURES**

- Only one layer of adhesive is necessary to achieve a full-strength bond
- Peel strength is significantly higher (p. 0.001) for tissueLINK™ than for leading competitor
- Lower heat during polymerization makes it more comfortable during application

**tissueLINK™ Flex Features**

All of the above features, plus:


- Flexibility allows closure of incisions up to 25cm
- tissueLINK™ Flex has been designed to combine flexibility and strength to close and protect wounds
- Special applicator tip provides better control and precision application
- Flexibility of glue adapts easier to the contours of the patient

tissueLINK™ directly compares to current market leader

**PACKAGING**



**BOX**  
10 Ampoules  
10 Applicator Tips

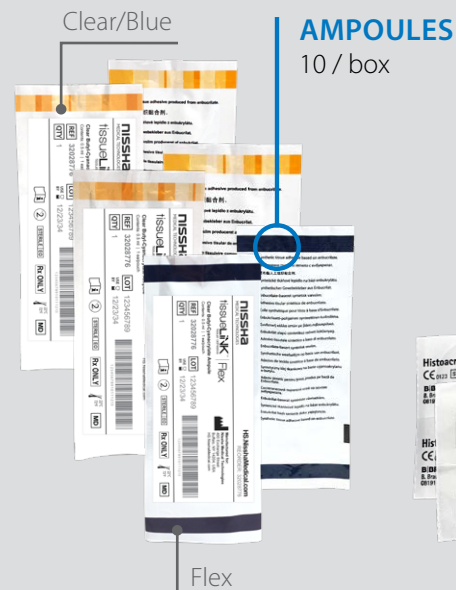

## MICROBIAL BARRIER

tissueLINK™, a n-butyl-2-cyanoacrylate allows **incisions up to 25 cm to be closed**. In vitro tests show that as long as the adhesive film remains intact, tissueLINK™ offers a **100% effective microbial barrier** for 7 days, against 7 different types of microorganisms, reducing wound infections.

Part #	Cross Ref.	Description	UOM	#/UOM
32031775	TS1051260-P	tissueLINK™ Flex 10 / 0.5 mL ampoules   10 Applicator Tips	Box	10
32031774	TS1051260FP	tissueLINK™ Blue 10 / 0.5 mL ampoules	Box	10
32031771	TS1050044FP	tissueLINK™ Clear 10 / 0.5 mL ampoules	Box	10

## tissueLINK™ FLEX INSTRUCTIONS FOR USE

### OPEN



Open the package and take out the tissueLINK™ Tip.

### PREPARE APPLICATOR TIP



Open the ampoule by twisting off the ribbed tip. Attach the tissueLINK™ Tip to opened ampoule.

### APPLY



Apply the glue in a thin layer to the approximated wound edges and hold in apposition for 60 seconds.

**The polymerization starts immediately upon contact with tissue.**

## tissueLINK™ COMPARISON

	tissueLINK™ Clear, Blue	tissueLINK™ Flex
<b>Composition</b>	n-butyl-cyanoacrylate	n-butyl-cyanoacrylate with Plasticizers
<b>Color</b>	<ul style="list-style-type: none"> <li>Violet (D&amp;C violet n°2)</li> <li>Clear</li> </ul>	<ul style="list-style-type: none"> <li>Violet (D&amp;C violet n°2)</li> </ul>
<b>Indications</b>	Skin Closure	Skin Closure
<b>Application</b>	Direct from ampoule	Direct from ampoule
<b>Color Code (packaging)</b>	White and Orange	Dark and Light Blue
<b>Storage</b>	24 months @ 22°C	24 months @ 25°C